


REDGUM BRAND
Daily Living and
Lifestyle Products

HEIGHT ADJUSTABLE SEAT WALKER

RG4209 (6" WHEELS)

RG4210 (8" WHEELS)

Edition 1

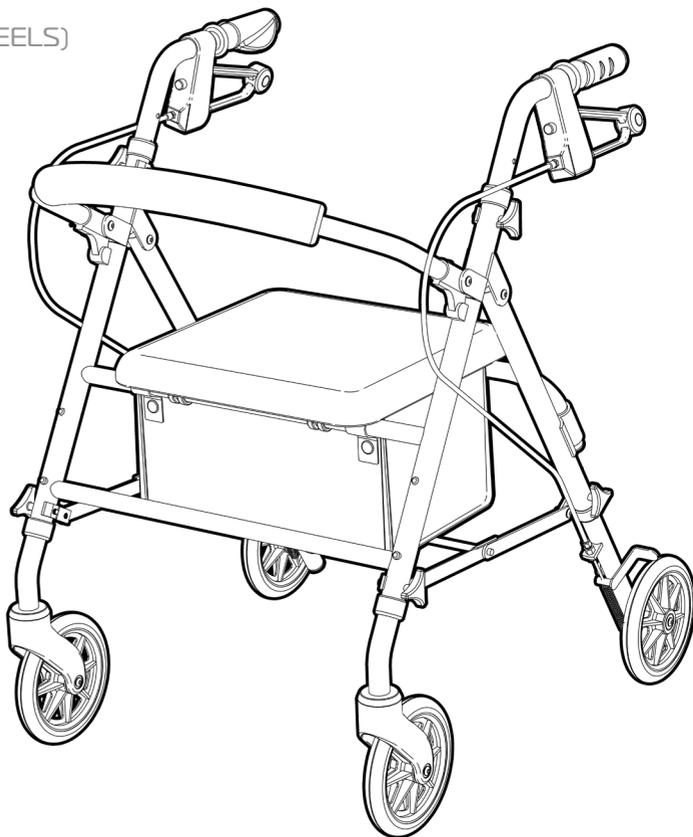


USER MANUAL

Please read and keep these instructions

ASK FOR THE BRAND

redgumbrand.com.au

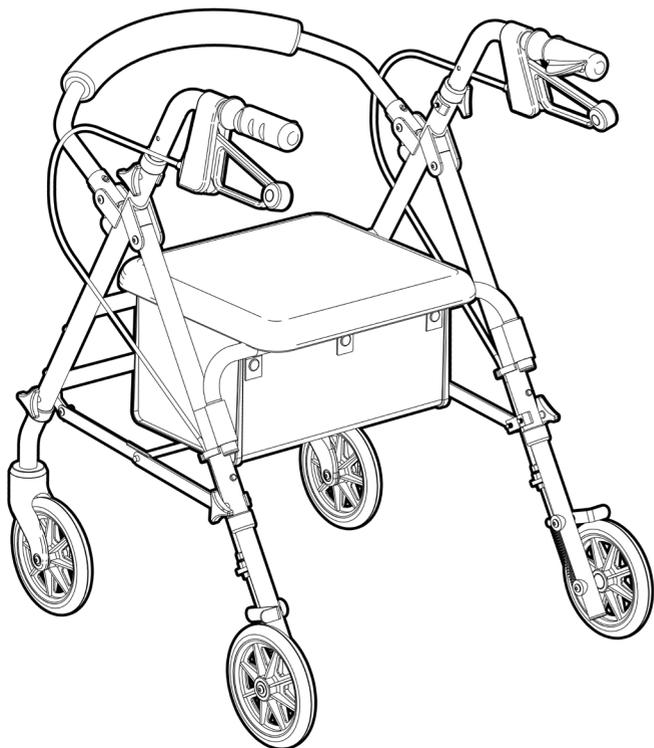

REDGUM BRAND

Daily Living and Lifestyle Products

REDGUM Brand owns the copyright to the Height Adjustable Seat Walker user manual. Any information contained in this user manual shall not be photocopied, reproduced or translated into any other languages without the prior written consent of REDGUM Brand.

REDGUM Brand strives to provide the highest quality products possible. Products are subject to change without notice.

© Copyright **REDGUM Brand**




REDGUM Brand is part of **FOR-DE Group Pty Ltd**

ASK FOR THE BRAND

redgumbrand.com.au

CONTENTS

1. **INTRODUCTION** - INTENDED USE
2. **PRODUCT OVERVIEW** - FEATURES / SPECIFICATIONS
3. **ASSEMBLY** - UNPACK and INSPECT SUPPLIED PARTS
4. **ASSEMBLY** - UNFOLD WALKER
5. **ASSEMBLY** - FIT FRONT WHEELS
6. **ASSEMBLY** - FIT REAR WHEELS
7. **ASSEMBLY** - FIT HANDLES
8. **ASSEMBLY** - HANDLE HEIGHT
9. **USAGE GUIDELINES** - BRAKING
10. **USAGE GUIDELINES** - SAFETY PRECAUTIONS
11. **PRODUCT CARE** - INSPECTIONS / CLEANING
12. **WARRANTY**

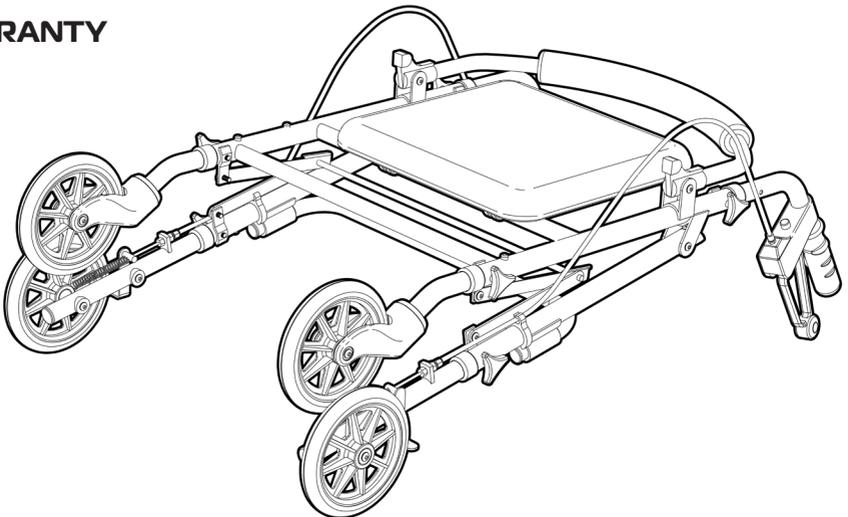

INTRODUCTION

Thank you for purchasing this REDGUM Brand product.

If you have any issues or questions about the Height Adjustable Seat Walker please contact your supplier or REDGUM Brand direct:

T: **+61 8 9248 4180**

E: **orders@for-de.com.au**

A: **1 Business Way, Malaga, Western Australia 6090**

INTENDED USE

This product has limitations and should be used in accordance to these guidelines. Failure to follow these guidelines may result in damage to the product and injury to the user. It is the responsibility of the user to ensure the suitability of this product for its intended use. The user should consult their appropriate healthcare professional if unsure of the correct product usage for their unique circumstances.

Only use the Height Adjustable Seat Walker for its intended purpose which is as a walking and sitting aid that can adjust to suit the users individual requirements.


Suitable for indoor and outdoor use.

IMPORTANT NOTE

This equipment **MUST** be assembled by a competent person.

Maximum User Weight: 120kg

Exceeding the weight limit could cause damage to the product and could put the user at risk.

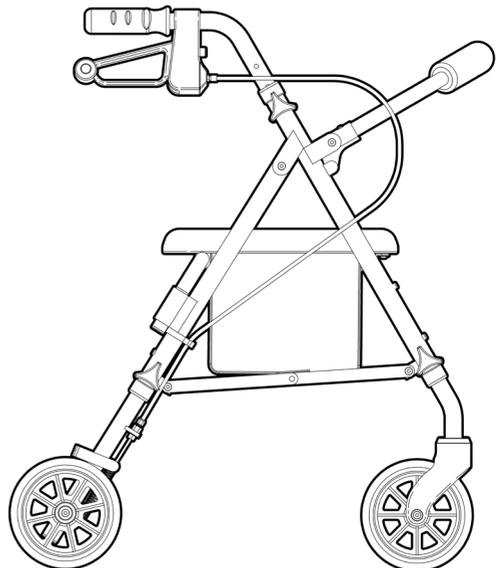

1.


ASK FOR THE BRAND

redgumbrand.com.au

PRODUCT OVERVIEW

FEATURES

- Great walking and stability aid
- Height adjustable seat and handles
- User friendly brakes / park brakes
- Ergonomic handgrips
- Comfortable padded seat
- Storage bag underneath seat
- Padded backrest
- Large diameter wheels for excellent mobility
- Flat free tyres
- Folds up for transit or storage

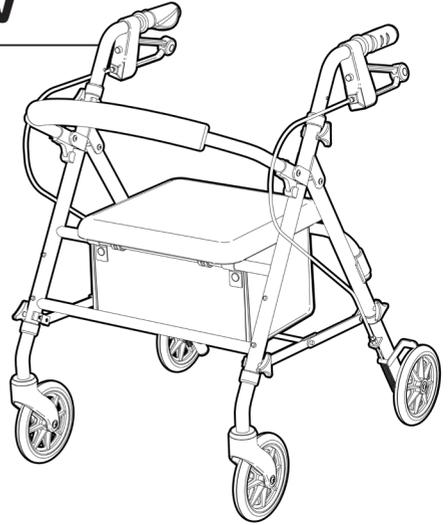

SPECIFICATIONS (All dimensions shown are nominal and at lowest setting)

RG4209 (6" WHEELS)

- Overall Width: **580mm**
- Internal Width: **440mm**
- Depth Unfolded: **625mm**
- Depth Folded: **290mm**
- Overall Height: **730mm** (lowest setting)
- Handle Height: **730 - 860mm** (8 settings)
(Based on the low seat height setting)
- Handle Height: **800 - 930mm** (8 settings)
(Based on the high seat height setting)
- Seat: **350mm (W) x 330mm (D)**
- Seat Height: **500mm (Low) / 560mm (High)**
- Wheel Diameter: **Ø150mm / 6"**
- Product Weight: **6.7kg**
- Maximum User Weight: **120kg**


RG4210 (8" WHEELS)

- Overall Width: **600mm**
- Internal Width: **440mm**
- Depth Unfolded: **640mm**
- Depth Folded: **340mm**
- Overall Height: **760mm** (lowest setting)
- Handle Height: **760 - 900mm** (8 settings)
(Based on the low seat height setting)
- Handle Height: **830 - 960mm** (8 settings)
(Based on the high seat height setting)
- Seat: **350mm (W) x 330mm (D)**
- Seat Height: **530mm (Low) / 590mm (High)**
- Wheel Diameter: **Ø200mm / 8"**
- Product Weight: **7.7kg**
- Maximum User Weight: **120kg**

Please contact REDGUM Brand if you would like any further information.

ASSEMBLY - UNPACK and INSPECT SUPPLIED PARTS

Ensure you have been supplied with the following parts:

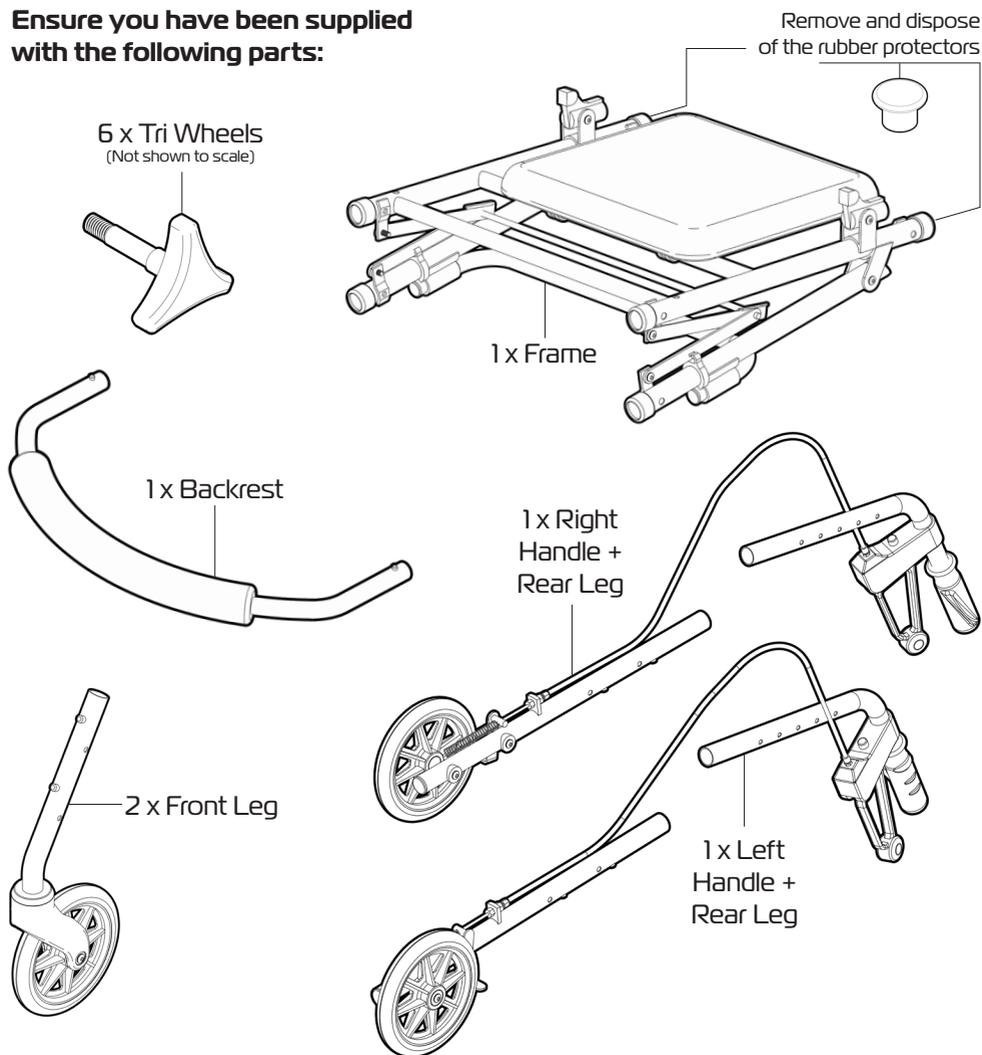

Please take care when unpacking the product and removing the protective packing materials.

IMPORTANT NOTE

Check the product for any visible damage. If you see any damage or suspect a fault, **DO NOT USE THE PRODUCT** and contact your supplier for support.

3.



ASK FOR THE BRAND

redgumbrand.com.au

ASSEMBLY - UNFOLD WALKER

Please follow these steps to unfold the seat walker frame correctly:

1. Position the frame on its side as shown.
2. Rotate the rear legs on frame out.
3. Press down on the leg braces on both sides to ensure the frame has fully opened out.
4. Gently rotate seat down onto frame.

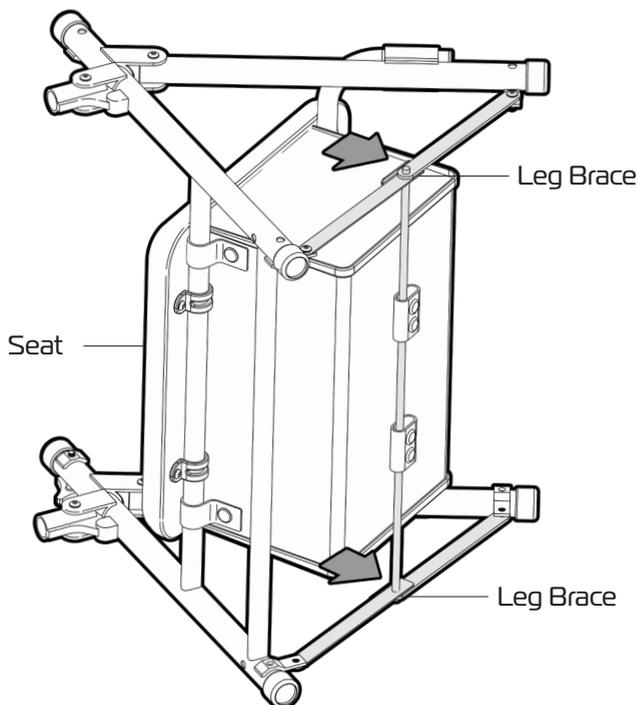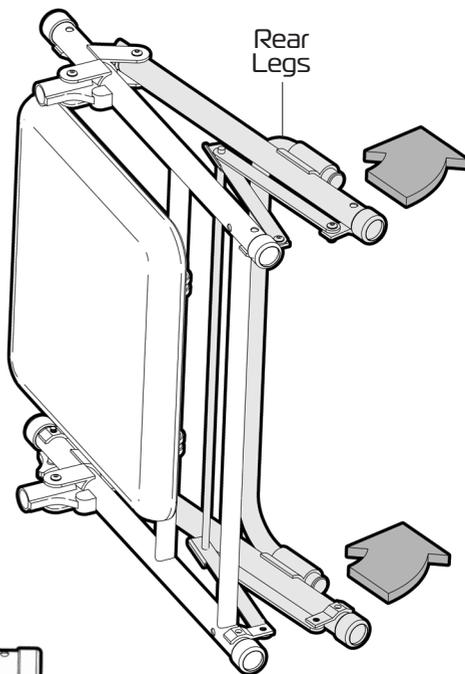

ASSEMBLY - FIT FRONT WHEELS

Please follow these steps to fit the front wheels onto the seat walker frame correctly:

1. Line up the tube on front leg with the tube on the frame as shown. (Press buttons must line up with hole on front of main frame)

2. Slide the wheel tube into the frame tube.


3. Push in the 1st press button and continue to slide the tube into the frame.

4. Keep pushing the leg in until the press button head pops out of the hole in the frame.


If the lower seat height setting is required, push in the 2nd press button and continue to slide the leg in until the press button head pops out of the hole in the frame.

5. Wind in the tri wheel to lock the wheel in place.

6. Repeat the process to fit the other front leg to the frame.

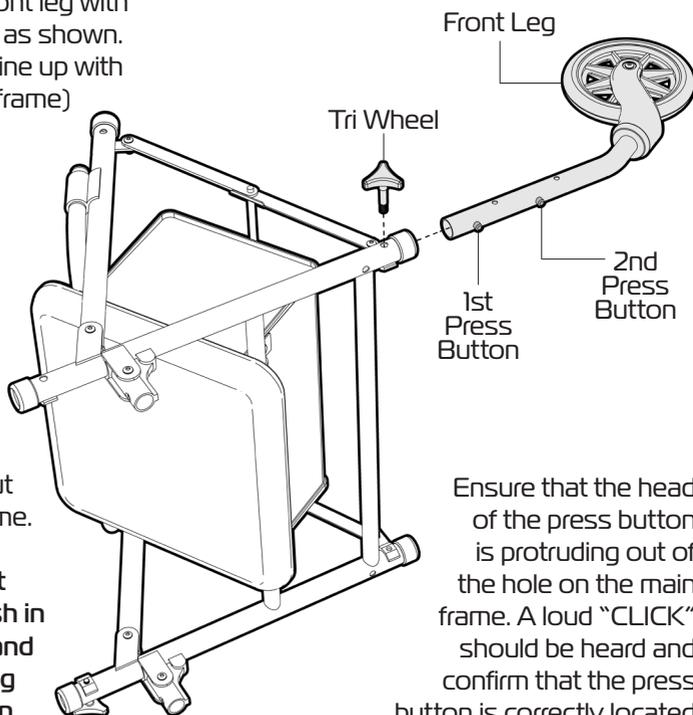

Ensure that the head of the press button is protruding out of the hole on the main frame. A loud "CLICK" should be heard and confirm that the press button is correctly located in place.

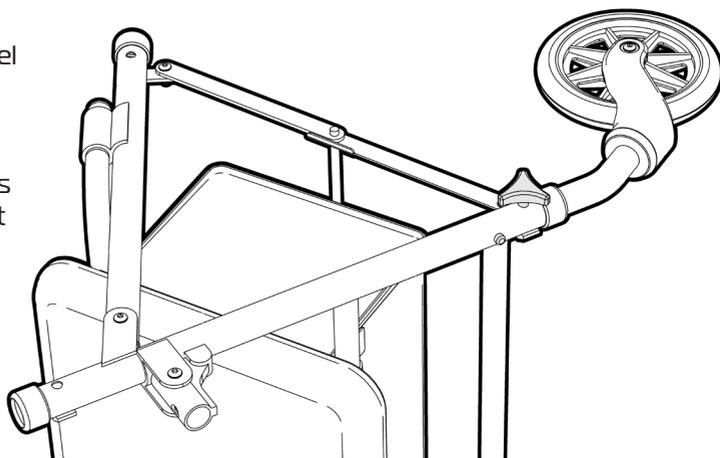

5.

ASK FOR THE BRAND

redgumbrand.com.au

ASSEMBLY - FIT REAR WHEELS

Please follow these steps to fit the rear wheels onto the seat walker frame correctly:



Determine the correct leg and handle to go on the correct side of the frame.

Wheel must face toward the outside of the main frame.

1. Line up the tube on rear leg with the tube on the frame as shown. (Press buttons must line up with hole on rear of main frame)
2. Slide the wheel tube into the frame tube.
3. Push in the 1st press button and continue to slide the tube into the frame.
4. Keep pushing the leg in until the press button head pops out of the hole in the frame.

If the lower seat height setting is required, push in the 2nd press button and continue to slide the leg in until the press button head pops out of the hole in the frame.


5. Wind in the tri wheel to lock the wheel in place.
6. Clip the brake cable into the clip on the frame.
7. Repeat the process to fit the other front leg to the frame.

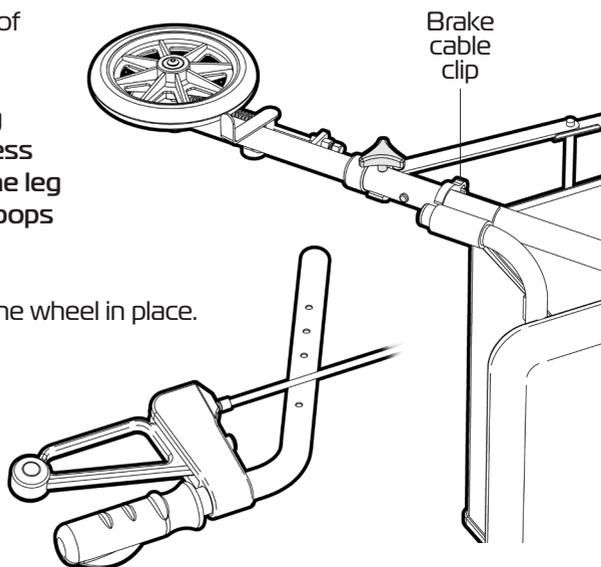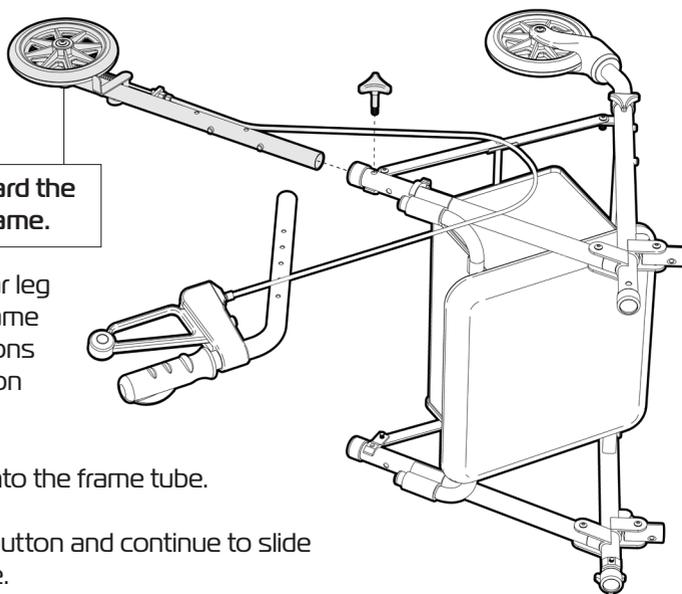

6.

ASSEMBLY - FIT HANDLES

Please follow these steps to fit the handles onto the seat walker frame correctly:

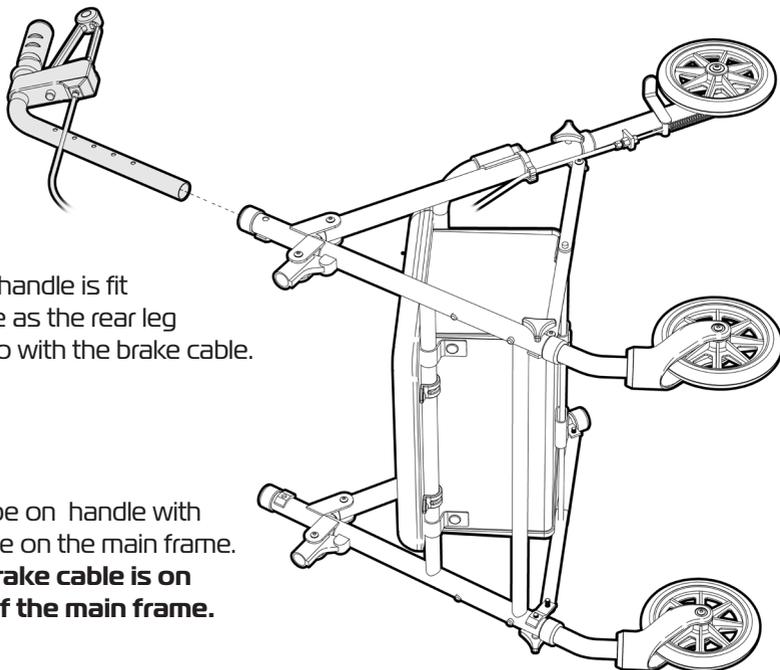


Ensure that the handle is fit to the same side as the rear leg it is connected to with the brake cable.

1. Line up the tube on handle with the correct tube on the main frame.

Ensure the brake cable is on the outside of the main frame.

2. Slide the handle tube down into the frame tube.

Refer to next page for information on handle height settings and fixing the handles in place.

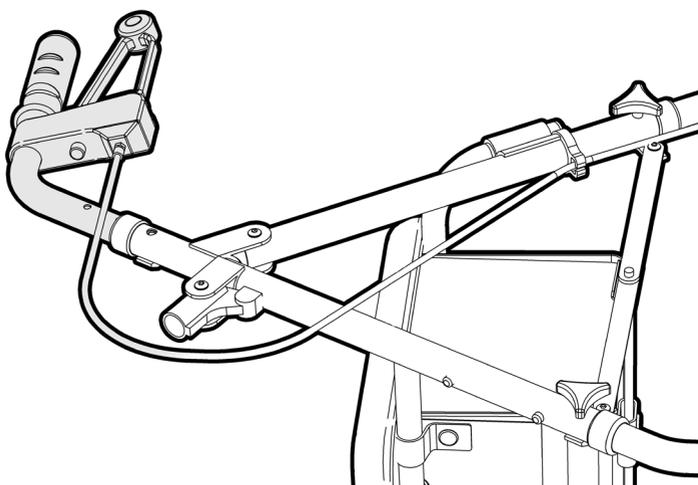

7.

ASK FOR THE BRAND

redgumbrand.com.au

ASSEMBLY - HANDLE HEIGHT



It is very important that the handles on the Height Adjustable Seat Walker are set to the correct height to suit the user.

Stand behind the seat walker with your shoulders relaxed and your arms hanging loosely by your sides.

The height of the handles should be adjusted so that they are level with your wrists.

This will position the arms at the recommended 20 degree bend in the elbow when using a seat walker.

1. Align the closest hole in the handle tube with the hole in the side of the main frame.
2. Wind in the tri wheel to lock the handle in place.
3. Repeat the process to fit the other handle to the frame.

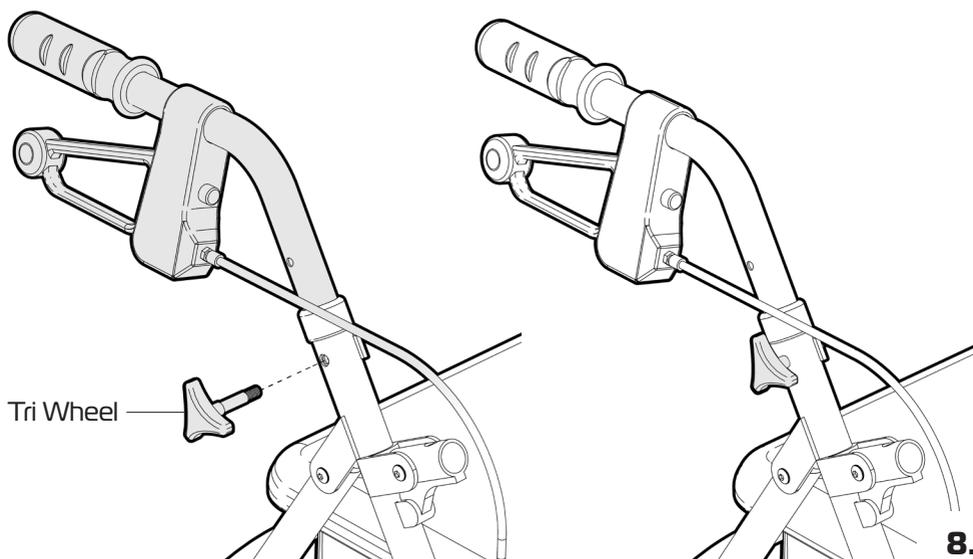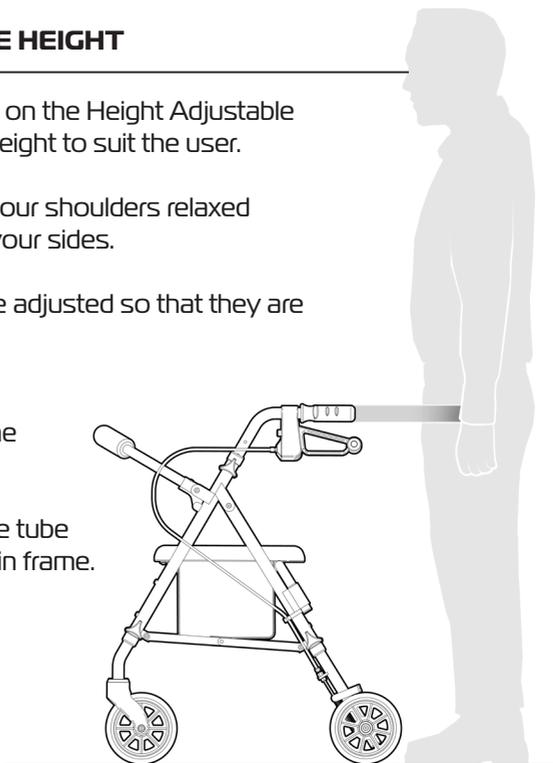

USAGE GUIDELINES - BRAKING

The Height Adjustable Seat Walker has independent hand brakes on both handles.

IMPORTANT NOTE:


Both hand brakes should be used simultaneously to ensure the seat walker stops as required and the user is in full control.

SLOW DOWN AND STOP

To slow down or stop the seat walker from rolling:

Gently pull up on both hand brakes simultaneously.

Release both hand brakes to allow the seat walker to move again.

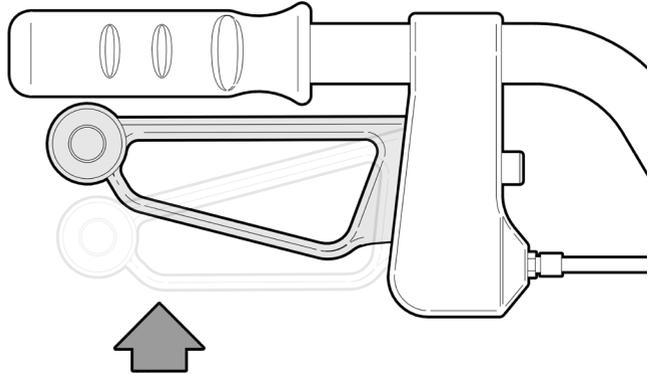

PARK BRAKE


To engage the park brakes on the seat walker:

Gently push down on both hand brakes simultaneously until a loud click is heard.

This will stop the seat walker from rolling.

To release park brakes:

Pull up on both hand brakes simultaneously until a loud click is heard.

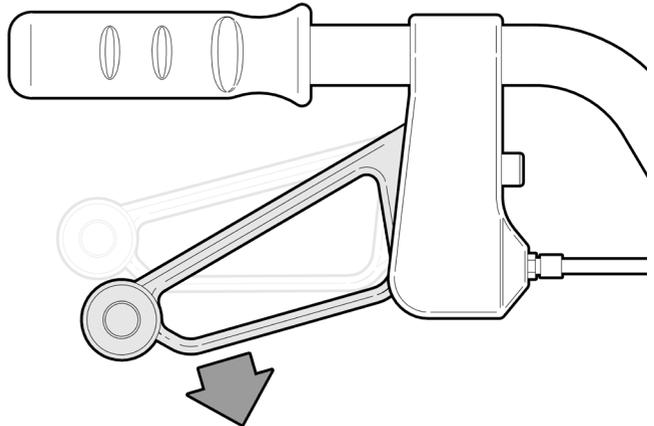

9.

ASK FOR THE BRAND

redgumbrand.com.au

USER GUIDELINES - SAFETY PRECAUTIONS

Please follow these guidelines to get the optimal benefit from the Height Adjustable Seat Walker. Failure to follow these guidelines may result in damage to the product and put the user at risk.

The Height Adjustable Seat Walker is designed to take a maximum load of 120kg.

- Do not overload the seat walker.
- Do not overload the bag.
(5kg maximum loading)

Pinch points do exist on this product. Please keep hands and any other body parts away from moving parts when folding the seat walker.

We recommend that the park brakes are only engaged when the user is getting on or off the seat walker.

Please take extra care when taking the seat walker offroad.

Go up or down inclines directly. Do not try to cross on an angle as this will cause instability and the seat walker may tip over.


All four wheels should always remain on the ground when possible.

The seat walker is not intended to be self propelled while seated.

IMPORTANT NOTE

Do not fit any parts or accessories to the Height Adjustable Seat Walker that are manufactured by another company. To do so will void your warranty and may seriously compromise your safety.

If you have any issues or questions about the product please contact your supplier or REDGUM Brand direct.

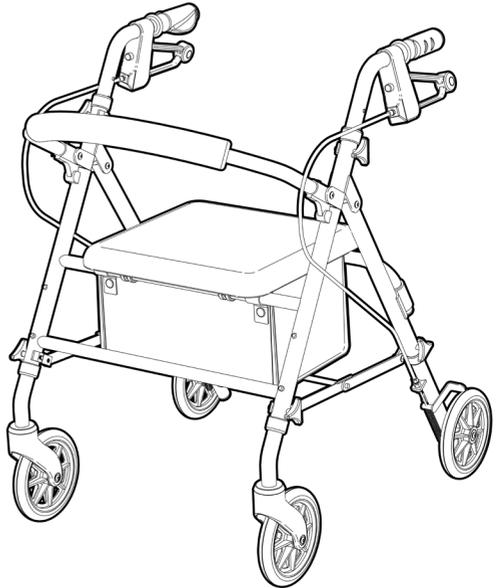

PRODUCT CARE - INSPECTIONS / CLEANING

INSPECTIONS

We suggest that you visually check the Height Adjustable Seat Walker periodically for signs of damage or wear prior to use, paying particular attention to the following areas:

Framework: Check for damage or distortion.

Wheels: Check to ensure the seat walker rolls as required.

Brakes: Check to ensure both brakes work as required (brake and park brake).

Hand grips: Check for cracks / tears in material.

Overall check that no components are loose or worn and that the seat walker operates as required.




Withdraw the seat walker immediately if you suspect damage or any other fault. If in any doubt, please do not issue or use and contact your supplier for service support.

CLEANING

Clean the Height Adjustable Seat Walker using a non-abrasive cleaner or mild detergent with a soft cloth.

Abrasive cleaners or abrasive cleaning pads could damage the seat walker and should not be used.

If you have any questions about the cleaning of the seat walker please contact your supplier or **REDGUM Brand** direct.

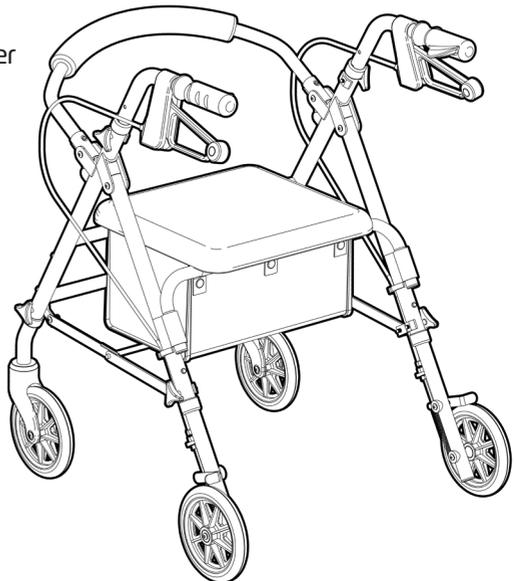

WARRANTY

The REDGUM Brand:

RG4209BL: Height Adjustable Seat Walker (Blue)

RG4209PS: Height Adjustable Seat Walker (Pewter Silver)

RG4210BL: Height Adjustable Seat Walker (Blue)

RG4210PS: Height Adjustable Seat Walker (Pewter Silver)

is covered by a 12 Month Warranty from the date of purchase.

If the product has any manufacturing faults during this period contact the place of purchase for assessment. Failures as a result of **ACCIDENTAL MISUSE, MODIFICATION OR WEAR AND TEAR ARE NOT COVERED BY THIS WARRANTY.**

During the guarantee period, if the company is satisfied there is a genuine fault, we will repair or replace the product at our earliest convenience. There may be freight charges if product needs to be returned to your place of purchase.

In order for this guarantee to be valid, you must:

- Be the original purchaser
- Provide a copy of the original invoice.
- Comply with manufacturer care and maintenance instructions.

Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement part for a major failure and for compensation for any other reasonably foreseeable loss or damage. You are entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and failure does not amount to major failure.

Do not modify the seat walker in any way or fit any parts or accessories that are not specified as an option for the product. To do so will void your warranty and may seriously compromise your safety.

If you have any issues or questions about the seat walker please contact your supplier or REDGUM Brand direct on:

T: **+61 8 9248 4180**

E: **orders@for-de.com.au**

A: **1 Business Way, Malaga, Western Australia 6090**

DISTRIBUTOR:

REDGUM BRAND
*Daily Living and
Lifestyle Products*



**Proudly
Australian Owned
and Operated**

ASK FOR THE BRAND

redgumbrand.com.au