
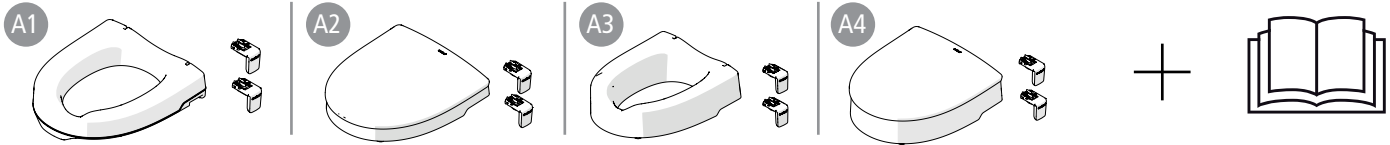
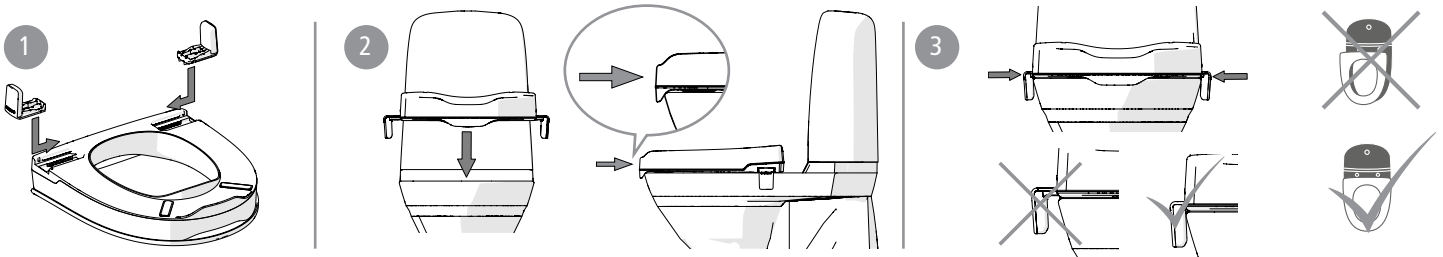
A

B

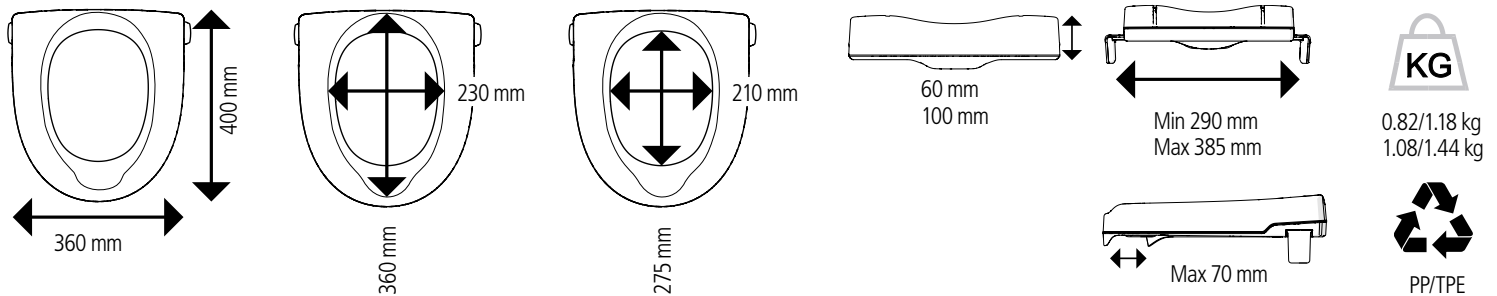C

D

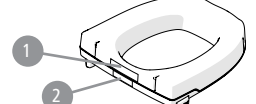
E

F

G

sv no da en de fi fr nl is it es pt

Etac My-Loo med kantstopp

Allmänt

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Etac My-Loo with brackets

General

Thank you for choosing an Etac product. In order to avoid damage during assembly, handling and use, it is important to read this manual and save it for future reference. You can also find it at www.etac.com. You can select your language via the "International" and "Local websites" link. Here you will also find other product documentation, such as prescriber information, pre purchase guide, and reconditioning instructions.

Intended use

The Etac My-Loo with brackets is a toilet seat raiser designed to facilitate toilet use by persons with restricted mobility. It is fitted directly onto the toilet, with the existing toilet seat lifted or removed. The Etac My-Loo is intended for use in home environments and similar.

Etac My-Loo may not be used in any other way than above stated.

Contraindications

A user with extensive balance problems should consider using a toilet seat raiser with arm support.

The product conforms to the requirements of the Medical Devices Regulation (EU) 2017/745.

 Do not stand on the product.

Delivery contents [Figure A](#)

Installation [Figure B](#)

1. Fit the brackets.
2. Place the toilet seat raiser on the toilet.
3. Press the brackets against the toilet to lock the toilet seat raiser. Make sure it is in the center of the toilet.

Technical data [Figure C](#)

The materials used are corrosion-resistant.

Maintenance [Figure D](#)

Cleaning and disinfection

1. Clean the product with normal non-abrasive cleaning agents (pH value of between 5-9). Rinse and dry. Disinfect the product with an alcohol-based disinfectant, 70%. The product is otherwise maintenance-free.
2. The product can withstand cleaning in a sterilizer at 85°C for 3 minutes. This applies for reconditioning.

The materials used are resistant to all general disinfectants.

Safety [Figure E](#)

1. Maximum user weight 190 kg.

 2. Defective or incorrectly installed products must not be used.

In case of an adverse event occurred in relation to use of the device, it should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to the manufacturer.

Guarantee, service life

5-year guarantee against material and manufacturing defects. For terms and conditions, see www.etac.com.

Service life 5 years. For complete information regarding the service life of the product, see www.etac.com.

The product's date of manufacture can be read from the barcode on the product. The number 11 is shown below the barcode in brackets. The number combination after these brackets is the date of manufacture.

Storage

The product should be stored indoors in a dry place at a temperature above 5°C. If the product has been stored for a long time (more than four months), its function must be checked by an expert before use.

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Etac Supply Center AB